

Caring for you

#### Contact Us:



www.neusoftmedical.com/en/



Neusoft Medical









### Boundless platform


Fiber Distributed Spectrometer



# Boundless platform Full-digital Gradient Chain

#### Full-digital gradient module


The gradient waveform generator is positioned in front of the gradient amplifier, forming a full-digital gradient module.

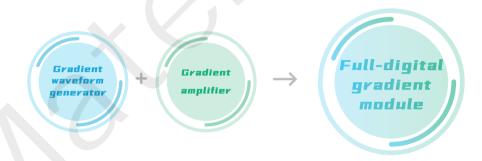

The realization of digital transmissionthrough optical fiber ensures both high bandwidth and fidelity of gradient output.

#### Advanced gradient system

The gradient system utilizes an embedded cooling design and real-time temperature control at multiple points to ensure more accurate temperature control and achieve accurate output of gradient waveforms.

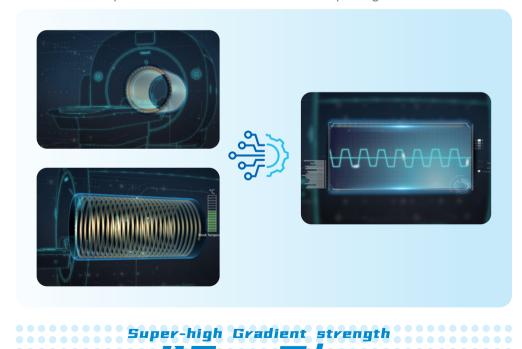




### High Resolution MUSIC-HOOWI Technology

High SNR/High Resolution/Artifact Free



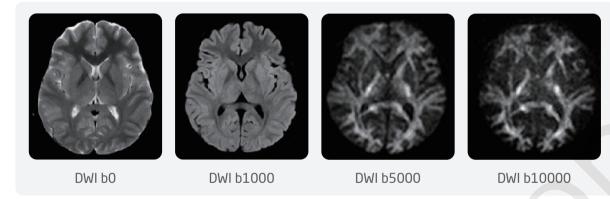

The higher b value, the higher performance requirements for the equipment. MUSIC sequence provides high spatial resolution and clear anatomical details.




### High Resolution


#### High Resolution

VAST can achieve 3D isotropic high-resolution imaging for T1, T2, FLAIR and PD, ensuring distortion-free MPR reconstruction. It can also be applied to various parts of the body.


Brachial plexus ▼

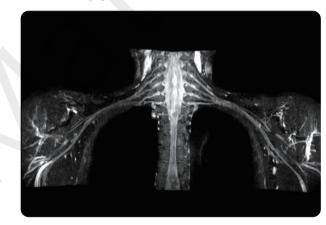


MRCP ▼

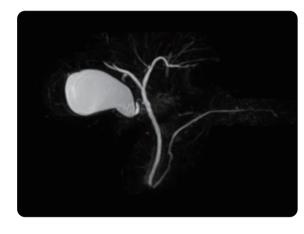

IAC ▼

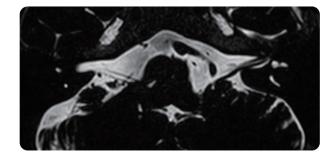

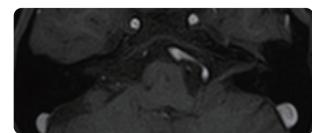

Joint ▼






## Ultra Fast Speed

Deep R-AI acceleration technology

Fast and fidelity

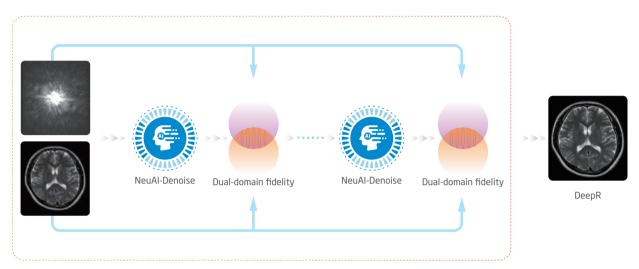

The dual-domain fidelity module ensures the authenticity of the data during the image reconstruction process as well as the stability of the reconstructed image.

Taking brain scanning as an example, compared to conventional scanning techniques, achieves the same image quality.

Time has been reduced


45%

### Ultra Fast Speed


Fast BrainQuant


Gradient echo formula

$$f(x) = \rho \cdot B1^{-} \cdot \sin(B1^{+} \cdot \theta) \frac{1 - \exp(-\frac{TR}{T_{1}})}{1 - \cos(B1^{+} \cdot \theta) \cdot \exp(-\frac{TR}{T_{1}})} \cdot e^{-\frac{TE}{T2^{*}}} \cdot e^{i(\gamma \Delta (\Delta B) \cdot TE + \varphi_{0})}$$

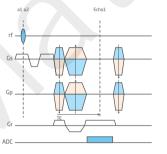

By optimizing the acquisition technique and mathematical algorithms, according to the gradient echo formula, multiple sets of equations are obtained by transforming the flip angle and echo time. By solving the equation groups, all quantitative values contained in the signals are obtained.

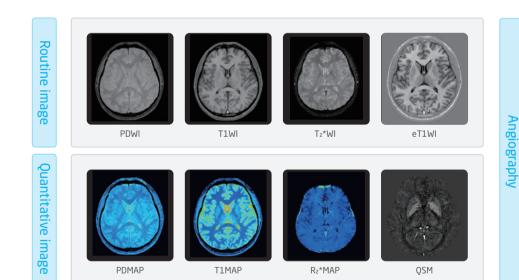

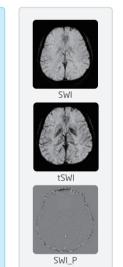

Faster


Accurate

Better




# Ultra Fast Speed


A single BH scanning can achieve ultra fast multi-phase dynamic enhancement scanning, which can continuously obtain arterial multi-subphase images, ensuring ultra-high temporal and spatial resolution.

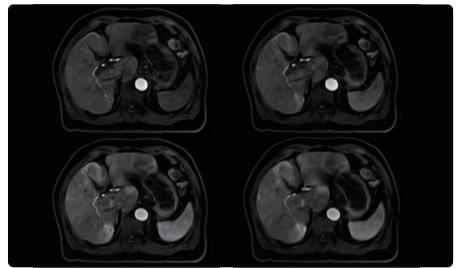

- ✓ Temporal resolution:4 sec/phase
- √ Spatial resolution:
- 0.7mm\*0.7mm\*2.5mm

DANCE adopts **K-space dynamic random sparse sampling**, K-space central area fully sampled repeatedly, and dynamic random sparse sampling of surrounding data.

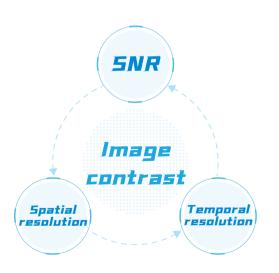


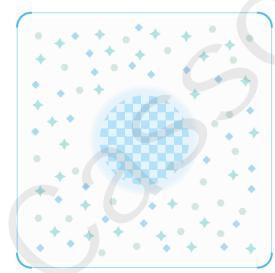

### Precise Quant

LiverQuant for Liver

Precise Quantification of Hepatic Fat & Analysis of Iron Deposition.

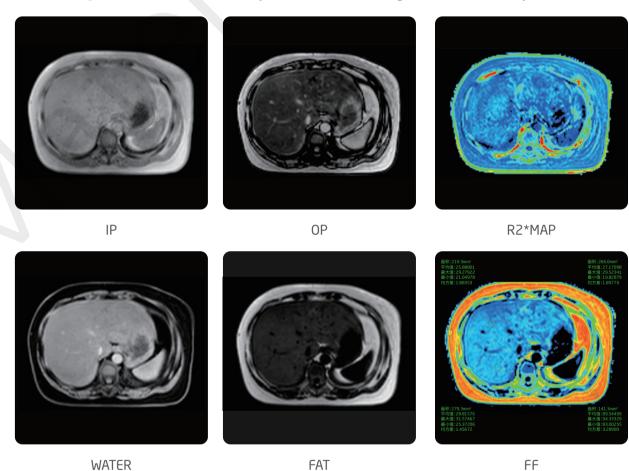

Six Echoes 1

「 One Acquisition 」

「Six Contrasts」

ा Precise Quant ।



Neusoft







### Precise Quant

#### KneeQuant

A noninvasive method to evaluate the biochemical changes of cartilage and meniscus in patients with early-stage osteoarthritis.

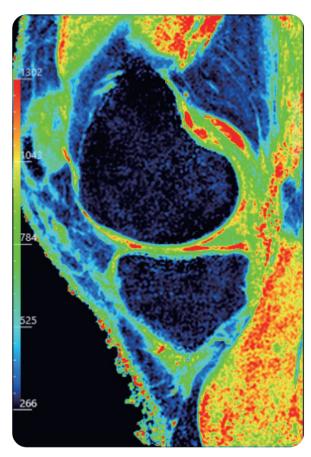

T1 Map

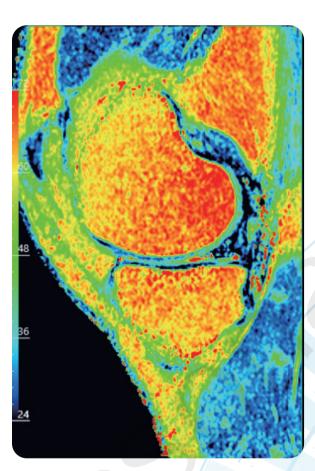


T2 Map

### NeuUI 2.0

Dual Screen Design

More convenient

More professional

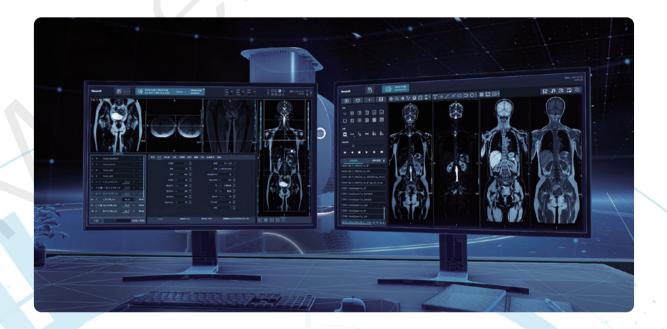






# Comprehensive Application

#### Nerves and Blood vessels ▼

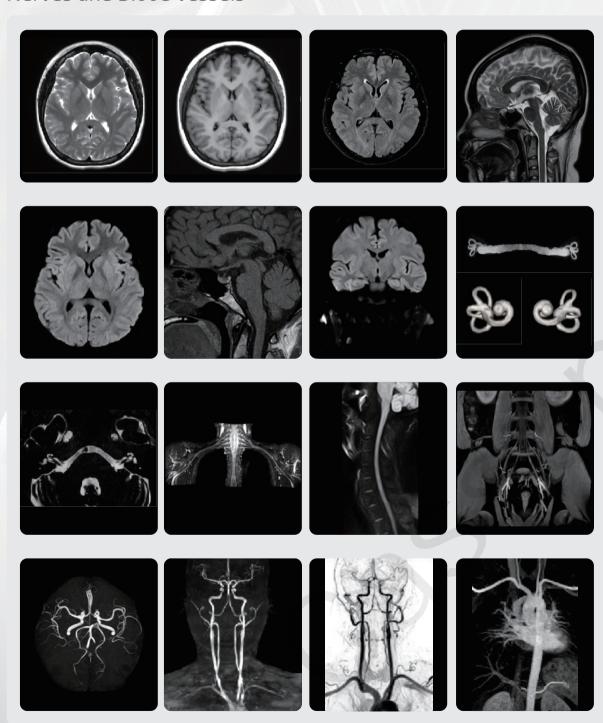


Joint ▼






# Comprehensive Application


Abdomen ▼

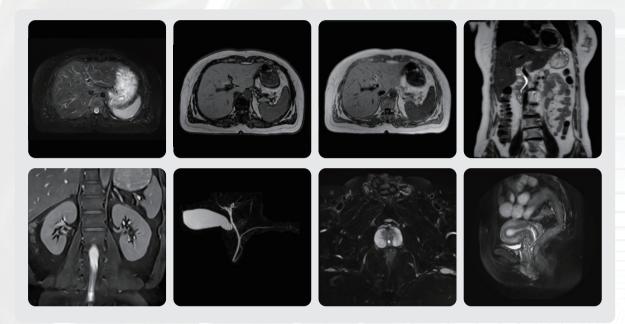

Total Spine & PET-LIKE ▼

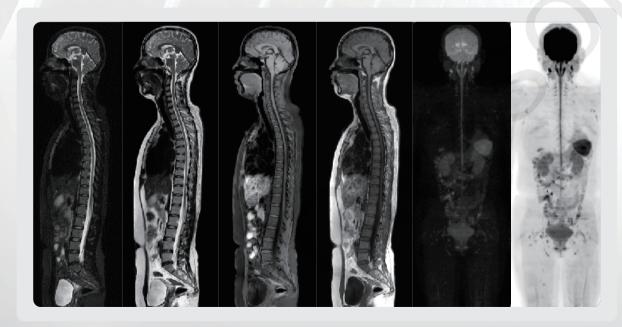

Deep R ▼

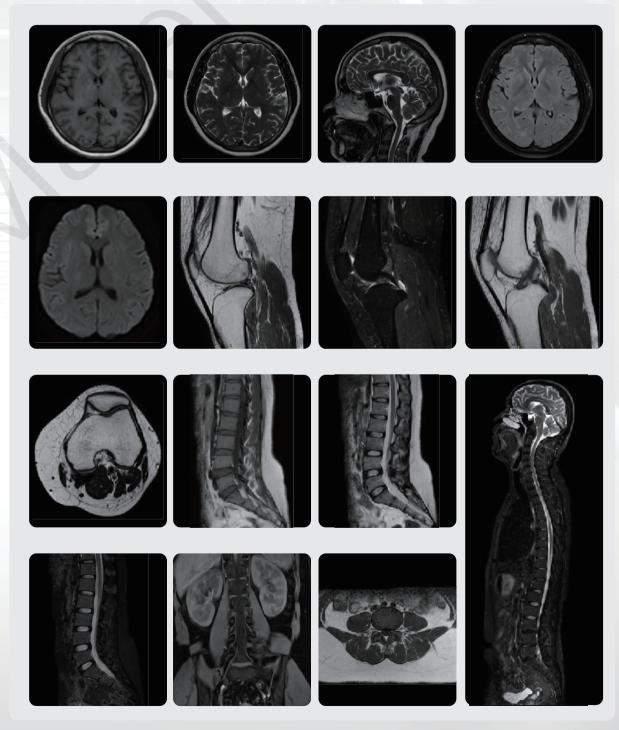






### Comprehensive Application




#### SWI ▼

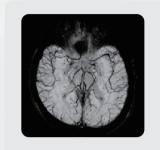


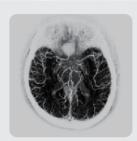

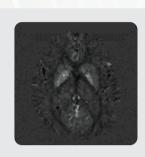


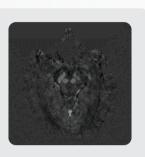

#### DTI ▼

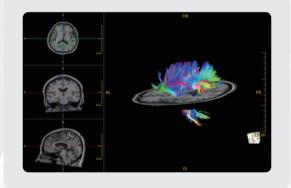



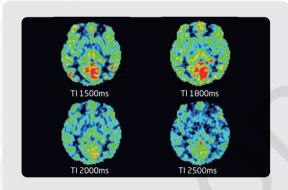

#### MRS ▼



### Neusoft Global Service & Logistics Network



#### After-sales service and support

- Remote service capabilities bring Neusoft expertise to you IMMEDIATELY, no matter where you are!
- Identifying and correcting PROMPTLY and PROACTIVELY, minimizing downtime and patient inconvenience.
- Global logistics network enables fast response regarding parts and supplies.
- \* Note: The contents of this publication and the listed parameters are for reference only and not intended as legal offers or commitments. Neusoft Medical Systems reserves the right to modify the contents, design, specifications and options described herein without prior notice, and will not be liable for any consequences resulting from the use of this publication. Please contact your local Neusoft sales representative for the current information. The specific sales product configuration is subject to the actual contract signed by Neusoft.

\* Not available in the United States.